




SLEEPCURITY FOR YOUR BABY

USER MANUAL
manual de instruções


bGuard components




Home Station

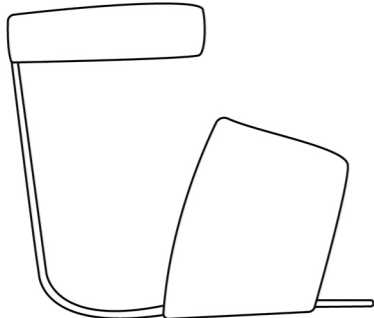

Smart Sock

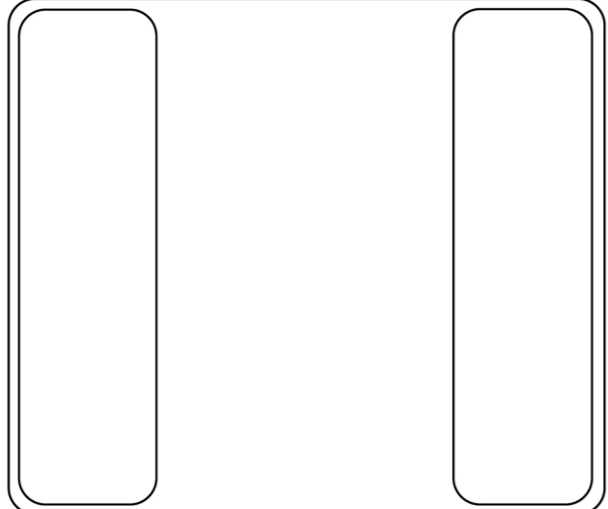


Smart Pillow

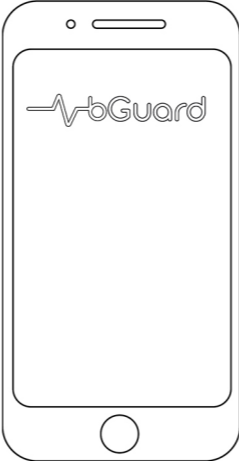

Mobile Application


bGuard charging

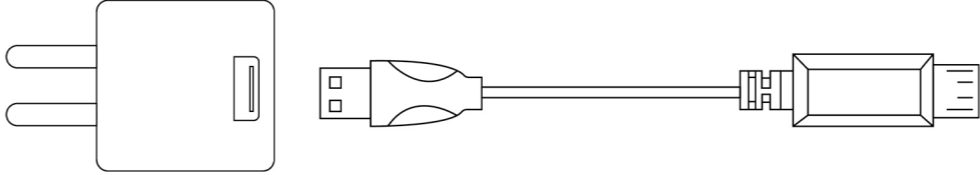

USB charger

USB



micro USB

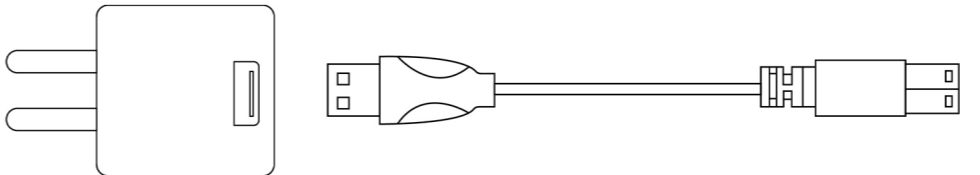

USB charger

USB

JST plug

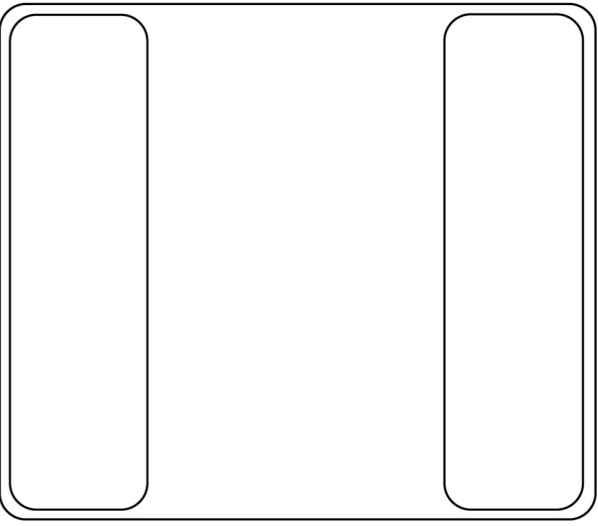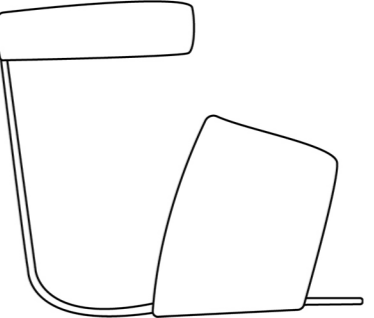

Limited Warranty and Disclaimer Information

LIMITED WARRANTY

SleepSense warrants that the bGuard will be free from defects in materials and workmanship, under normal use and service, for a period of two years from the date of retail purchase by the original purchaser.

If a hardware defect occurs and a valid claim is received, SleepSense will repair or replace the bGuard or a defective component at no charge. This only when the bGuard is in compliance with the terms of the Limited Warranty Period. It is important to know that this warranty does not apply to bGuards that have been damaged as a result of misuse, modifications or bad maintenance. When it is determined that it is not reasonable to replace the defective bGuard, a refund will be given to you equal to the purchase price paid for the product.

DISCLAIMER

The purpose of the bGuard is to help parents in taking care of their baby and letting them know the current state and health of their baby. It is a device meant for healthy babies and in no way a medical device. Furthermore it is not designed to prevent a Sudden Infant Death Syndrome. The bGuard is not developed to replace adult supervision.

By using the bGuard, the user agrees that SleepSense shall not be held liable for any loss, damage or injury whilst using the bGuard monitor, regardless of cause.

READ THIS MANUAL

This manual contains important information regarding the use of this product and safety regulations. Your safety and the safety of others is very important to us. Read this manual carefully and follow all the safety messages.

Important Safety Instructions

FOLLOW ALL INSTRUCTIONS




Do not use this device near water. To reduce the risk of fire or electric shock, do not expose the bGuard to rain or moisture.



Do not place near any heat sources such as radiators, heat registers, stoves, or other devices that produce heat.



Protect the power cord from being walked on or pinched particularly at plugs and the point where they exit from the device.

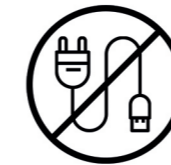


Unplug the device during lightning storms or when unused for long periods of time.



WARNING


Strangulation Hazard - Children have strangled in cords. Keep the cord from the home station out of the reach of children (more than 1 m away).

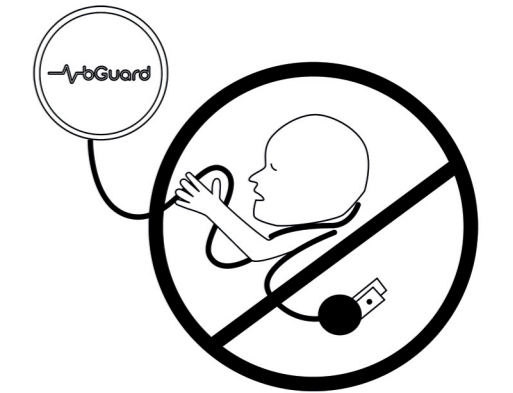

Putting on the Smart Sock

HOW TO ADJUST THE SMART SOCK TO YOUR BABY

- 1** Put the top band around the baby's ankle. Make sure it stays fixed but is not too tight, so blood flow is still possible. The Smart Sock can go on either the left or right foot.
- 2** Adjust the bottom band to the size of the baby's foot. The band (with sensor) should be placed in the middle of the sole of the foot.
- 3** Make sure that the sensor makes contact with the baby's skin. You can put clothes or a sock on top of the Smart Sock but do not place clothing between the band and the baby's skin.
- 4** The Smart Sock is now ready to keep track of your baby's pulse and sends the information to the bGuard mobile application.

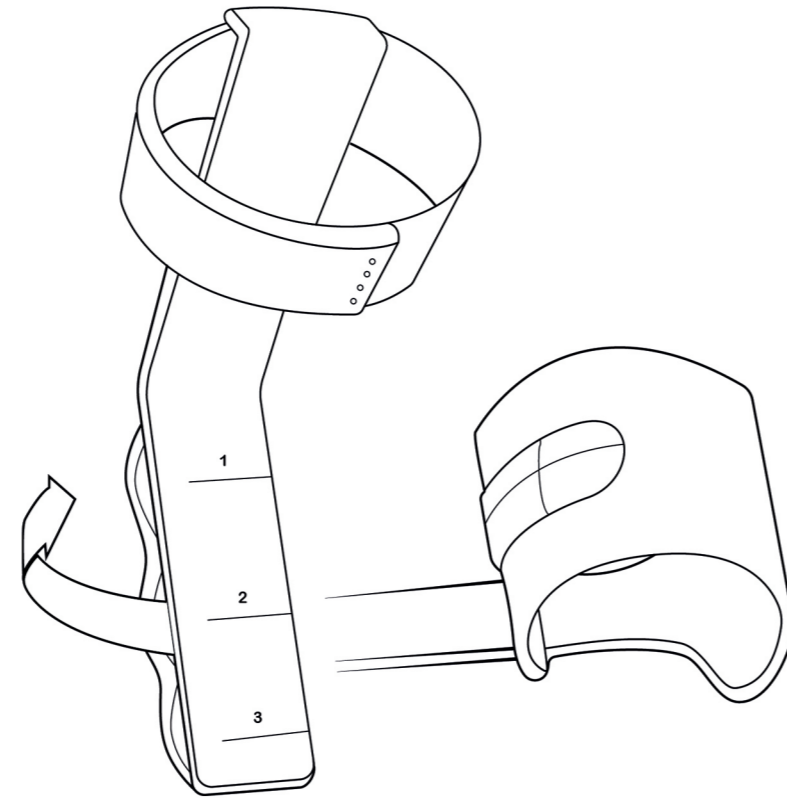

Handling and Recycling of your bGuard

TAKING CARE

Handle the bGuard components with care. The Home Station consists of plastic and contains sensors inside that can be damaged when not maintained properly.

The Smart Sock can be washed if desired. It is important to take out the sensor before washing the Smart Sock. The band can be hand washed or machine washed.

The Smart Pillow is also provided with a cover that can be removed and washed when required. The cover can also be hand or machine washed.



RECYCLING

Make sure that this product is not disposed with the household waste. This sign indicates that this product should not be disposed as household waste. Check your local authority for more recycling advice.

It is important that this product is recycled according to the selective sorting rules for electrical waste and electronic equipment.

bGuard complies with the directives concerning the Waste of Electrical & Electronic Equipment (WEEE). This means that this electronic device must be handled pursuant to European Directive 2002/96/EC in order to be recycled or dismantled to minimize its impact on the environment.



